


S T E R I L W I Z E . C O M



The most comprehensive automated system to achieve compliance with data validation, while saving time, reducing human error, and automatically generating required IPAC reports

S T E R I L W I Z E . C O M

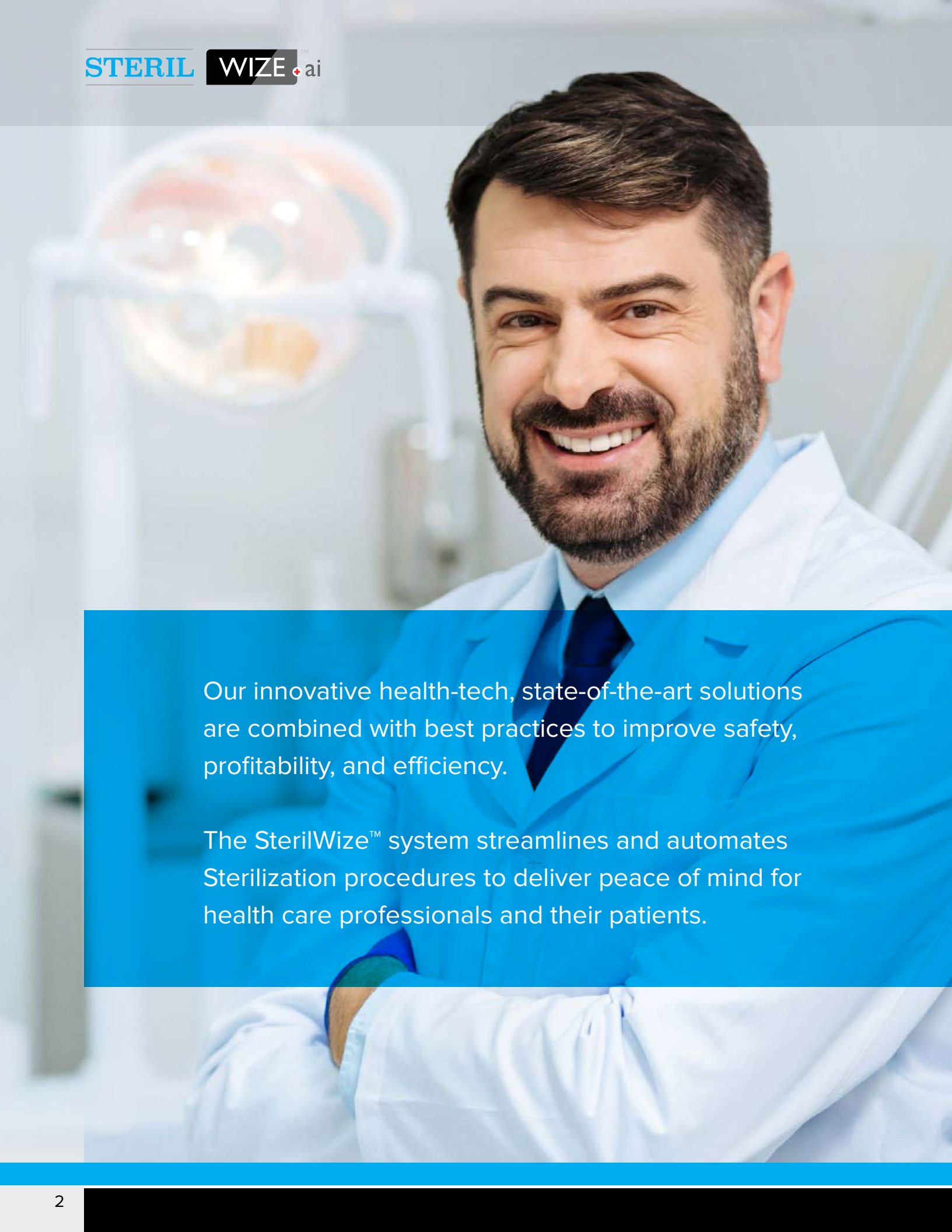

Our innovative health-tech, state-of-the-art solutions are combined with best practices to improve safety, profitability, and efficiency.

The SterilWize™ system streamlines and automates Sterilization procedures to deliver peace of mind for health care professionals and their patients.

What is SterilWize™?

SterilWize™ is a system that uses a proprietary and patented barcoding and tracking system to monitor instrument sets (cassettes or pouches) through the entire instrument reprocessing & sterilization cycle.

This allows SterilWize™ to identify each barcoded set of instruments from the beginning of sterilization through to the patient appointment where these instruments are used. Reports are generated in which each instrument set is associated with all of its related data such as:

- Date & time of sterilization
- Cycle number with Temperature & pressure data
- Operator
- Sterilizer
- BI test
- BD test
- EI test

When used with “Smart” sterilizers & “BI” incubators data is automatically collected and compiled into reports.

Therefore, at any time in the future, you will easily be able to determine which set of instruments were used on a specific patient, on a specific date, and have all related data related to that particular set of instruments.



“ When I first heard about the SterilWize™ system I assumed it was like other barcoded systems similar to the one our office already had. Fortunately, I agreed to test it out and my office staff found it far superior to our current system. Most significantly, it substantially reduces the time for documentation and the chance of operator error. As a bonus it offered us the opportunity to not only meet the standards in this area but also to surpass them by having individual sterilization records instantly uploaded to the patient record with one additional click.”

Dr. Jordan Soll

Sole Proprietor of Central Dental Group
Co-chairman to the Editorial Board of the Oral Health Journal
Dental Expert for Cityline TV
Central Dental Group | Toronto, Ontario

Sterilization Workflow Management System

Coaching

The SterilWize™ dashboard guides the staff member through the various steps of the sterilization process and if necessary can provide instant instructions and video demonstrations at any given step.

Monitoring

SterilWize™ ensures that each step in the Sterilization process is followed. The staff member performing each step of the process is automatically logged.

Data Collection and Storage

SterilWize™ automatically collects all necessary data and provides it to the clinic on a daily basis. Additionally, there is an encrypted cloud back-up of this data. The Sterilization records for each instrument set can also be linked directly to the specific patient appointment on which the instruments were used.

Custom Report Generation

Detailed sterilization records can be instantly produced for any given time period, specific appointments or based on any other criteria. These reports contain the data required by the various regulatory bodies & public health authorities to demonstrate compliance with the required standards.

Comprehensive Training

Onsite Training

As part of the installation process, a comprehensive training will take place for key office staff, followed by two weeks of close monitoring by SterilWize™ team members.

Remote Training

After the initial training, remote training is available 24/7 for any issues that may develop or questions that may arise. Remote training will also be available for upgrades to the system at no additional cost.


Video Training

Each office will be provided with a comprehensive training video embedded in the SterilWize™ software for continued reference or training of new staff members. In addition, the office will also receive a series of short videos embedded in the software for each step of the sterilization process. These videos can be immediately accessed through the tablet to provide a quick tutorial for that stage of the process.

Continuing Education

Core 1 IPAC Course

SterilWize™ has arranged to provide the University of Toronto's on-line Core 1 IPAC course available to all office staff at a reduced cost to allow all personnel to receive the necessary training and certificates that the standards require. These certificates can be uploaded to the staff section of the office IPAC manual staff as required by the regulations. SterilWize™ will also provide educational updates as necessary and discounts on other CE IPAC courses that it sponsors.

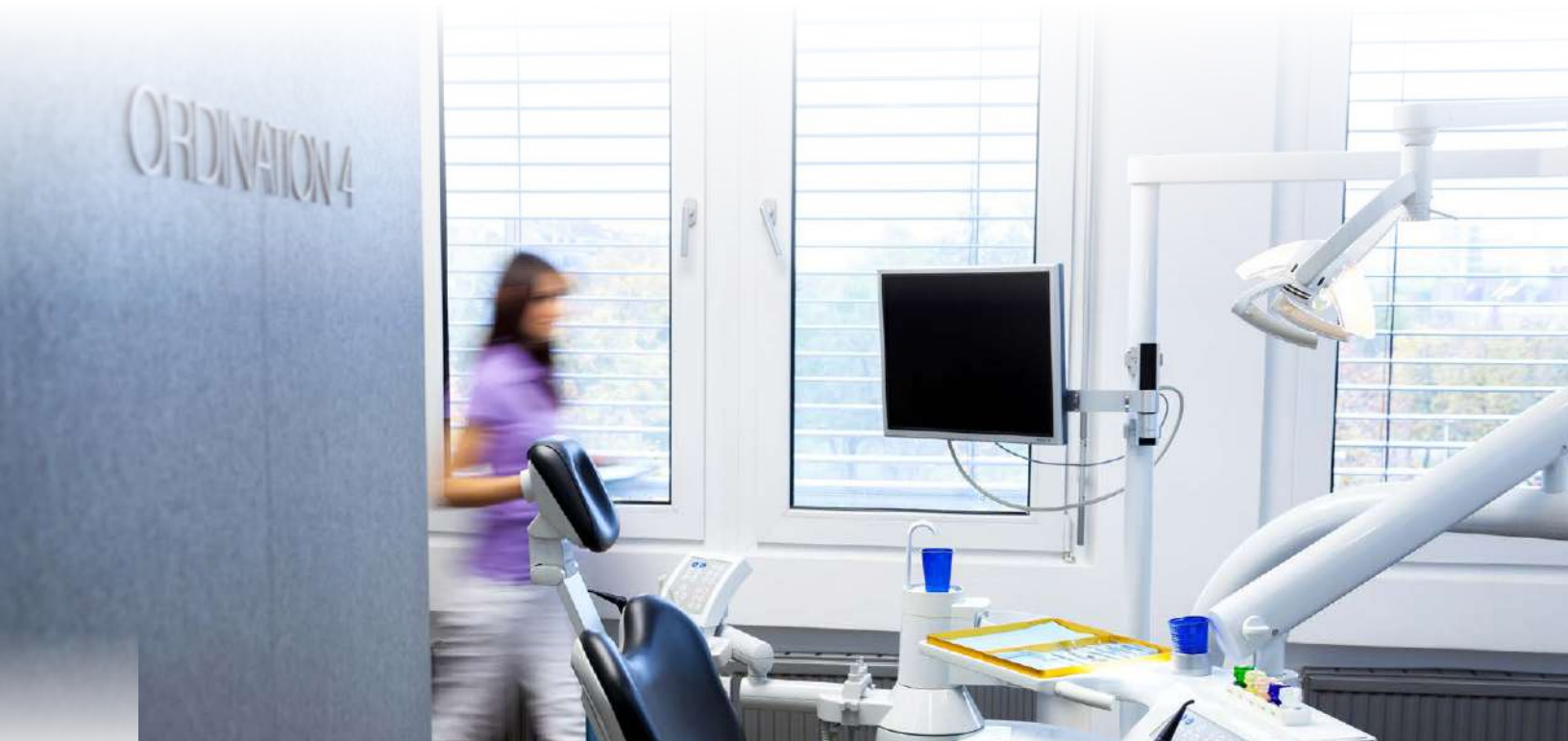

SterilWizeTM Customized Tablets

SterilWizeTM provides custom tablet computers whose dashboard is customized to the layout of your sterilization area and equipment. This dashboard is an interactive command center for all sterilization activities. The command center is organized by zones and colour coded to integrate the workflow and instructions exactly the way the staff member experiences the process.



STERILIZATION MONITORING

AUTOMATED DATA COLLECTION

AUTOMATED DOCUMENTATION

AUTOMATED REPORT GENERATION

ENCRYPTED CLOUD BACKUP OF ALL DATA

Assurance

First, it provides assurances to the clinic owner that the procedures are carried out properly in accordance with required standards.

Documentation

Instant digital documentation of all aspects of the sterilization process including staff member responsible, biologic indicators, the appropriate temperature, pressure and time specifications for the specific equipment used.

Compliance

Offers the opportunity to demonstrate conclusively to a patient, regulator, or public health authority that appropriate sterilization standards have been met.



“ I am very pleased to have invested in the SterilWizeTM process. My staff found it easy to learn and created significant time savings for them. I am happy to know that mistakes are not being made and I have the necessary documents to meet my regulatory requirements.”

Dr. Amelia Chan

Downtown Dentistry | Toronto, Ontario



PEACE OF MIND

STERIL

WIZE™ **ai**

1-866-224-WIZE

STERILWIZE.COM



“Several of our clinics have recently migrated from an analog data logging system to Sterill wize, and the results have been profound. The human error in instrument reprocessing was taken out of the equation. The digital logging capability significantly reduced the busy work in the steri-bay while eliminating the need to store masses of reprocessing logs. For dental clinics today, Sterilwize tracking software is a worthy investment in patient safety and peace of mind.

Julian Perez

V.P., Risk Management & Compliance
Dentalcorp.